

Let's get you drooling  
and on your way.



### Follow instructions

Don't eat, drink, chew gum or smoke within 60 mins of test—this can cause discoloration.

### Take your time

POOL saliva in your mouth, then DROOL onto the funnel. Don't spit — it can add bubbles and mucus.

### Check your work

Use a light source, or ask for help from site staff, to check your sample against these images.

### This is what an A+ sample looks like:

- Clear drool
- Correct volume (Between 1 and 1.5 mL)
- Cap is on tight



### You will have to repeat the test if your tube looks like this:

Discoloration



Thick mucus



Glob of mucus



Particulates



Wrong volume



Leaky tube

